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REF 4600 CE IVD

SAMPLING.pro

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SAMPLING.pro

For in vitro diagnostic use

Intended purpose: This product is intended for the self-collection of a solid stool sample using a swab, and for stabilizing the contained DNA in a pre-filled sample vial until subsequent laboratory analysis.

Intended user: This product is intended for sampling by a layperson in the home environment.

Explanation: The solid stool sample is collected by swabbing toilet paper with the sterile swab provided and transferring it into the sample vial. This kit is non-sterile and intended for single use only. The included reagent preserves the DNA in the unprocessed sample for up to four weeks at room temperature (8-30 °C) during transport. The microspheres in the reagent help distribute the sample evenly for optimal stabilization. This kit does not provide a test result. The stabilized sample is sent to a laboratory, where trained professionals analyze the DNA using molecular diagnostic methods.

Contents: Each product contains a sample vial with 0.75 mL stabilization reagent with microbeads, and a sterile packed swab.

Limitation: This product is NOT suitable for the collection of liquid fecal samples.

Warnings and precautions: Do not drink, touch, or remove the liquid reagent from the vial. Avoid contact with skin and eyes. If contact occurs, rinse thoroughly with water. If the kit is visibly damaged or the liquid has leaked, please use a new kit. Keep out of reach of children – small parts may pose a choking hazard.

Important: The product does NOT contain components in quantities that pose a health risk in the event of accidental contact.

Storage: 24 months at 8 - 30 °C prior to sampling. 30 days at 8 - 30 °C after sampling.

Disposal: Dispose of the used swab in its original packaging with your regular household waste. Any leftover material should be discarded according to local waste disposal regulations

Procedure
Step 1: Place the wrapped swab and sample tube on a flat, clean and dry surface.

Step 2: Unscrew the lid of the sample vial counterclockwise. Set the lid aside upside down for later use.

IMPORTANT: If you accidentally spill the stabilizing reagent, DO NOT refill the vial. Use a new product.

Step 3: Carefully open the sterile swab package and remove the swab without touching the tip. Use the swab to collect a small amount of stool from the toilet paper (or from supplied toilet accessories).

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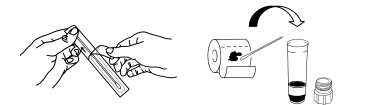
IMPORTANT: Take the stool sample free of urine or toilet water. AVOID using excessive amounts of sample. The sample MUST be completely covered with the stabilizing reagent to ensure the stability of the sample.

Step 4: Hold the open sample vial firmly in one hand. Carefully insert the tip of the swab with the stool sample into the vial until it is fully submerged in the liquid. To transfer the sample into the stabilization solution, gently rotate the swab 10 times between your thumb and index finger. Afterward, place the swab back into its original packaging and throw it away with your household waste.

Step 5: Use the lid from step 2 to securely close the sample vial. To do this, place the lid straight on the vial and turn it clockwise.

Step 6: Send the sample to the partner laboratory for analysis following the provided shipping instructions. For the best results, make sure to ship the sample within 3 days of collection.

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BIOMES NGS GmbH